ENGLISH

OMG™ Connector – M

Model SC-9310

Intended Use

The Observational Mechanical Gateway (OMG) Connector - M is used to connect the Precision® ETS to a single Medtronic® Snap-Lid Connector Cable (Model # 355031) or up to two Medtronic Twist-Lock Screening Cables (Model #3550-03).

The OMG Connector - M is for evaluation purposes only. It is an adjunct to, but not part of, the Precision System. Full trial of the Precision System should precede permanent implant.

CAUTION	
The OMG Connector is for use only under the direct supervision of a healthcare professional. The OMG Connector may not be sent home with the patient.	

Compatibility

The OMG Connector - M shall be compatible with the following Medtronic leads that connect to the OR Cable Assembly available prior to February 2007:

Model Name	Model Number
Pisces Quad™	3487A, 3887, 3888
Pisces Z-Quad™	3890, 3891, 3892
1x8 percutaneous lead	3776, 3777, 3778
8 contact trial lead	3873, 3874, 3875
Resume™ II & TL	3587A, 3986A
On-Point PNS™	3987A
SymMix™	3982A
Specify™ 2x4	3998
Hinged Specify™	3999

Pisces Quad, Pisces Z-Quad, Resume II, Resume TL, On-Point PNS, SymMix, Specify and Hinged Specify are trademarks of Medtronic, Inc.

Directions for Use

Connecting the OMG Connector

1. Inspect the OMG Connector for damage. If damaged, do not proceed. Return the OMG Connector to Boston Scientific Neuromodulation or your healthcare professional.

2. Turn off the non-Precision External Trial Stimulator (ETS).

CAUTION	
Failure to turn off the non-Precision ETS before connecting the OMG Connector may result in excess stimulation.	

3. Disconnect the proximal end(s) of the non-Precision OR Cable Assembly and insert into the corresponding port on the OMG Connector.
 - “Twist-Lock Screening Cables”(Medtronic Model # 3550-03)
 - * If the trial is using the Twist-Lock Screening Cables (MDT Model # 3550-03), plug the non-Precision OR Cable Assembly into the OMG Connector socket(s) labeled “1-L” (left) for a single-lead trial. The lead shows up as contacts 1-4 in the Bionic Navigator® software.
 - * For a two-lead trial, connect the non-Precision OR Cable Assemblies into “1-L” (left) and “2-R” (right). The lead shows up as contacts 1-4 and 9-12 in the Bionic Navigator software.
 - “Medtronic Snap-Lid Connector Cable” (Model # 355031)
 - * If the trial is using the Snap-Lid Connector Cable, plug the non-Precision connector into the OMG Connector socket labeled “Single Connector.” If 2 four contact-leads are placed in the Snap-Lid Connector Cable, the first and second leads show up as contacts 1-4 and 5-8, respectively, in the Bionic Navigator software. The 4-contact leads should be programmed in Manual Mode.
- * If 2 eight-contact leads are used, the leads show up as contacts 1-8 and 9-16 in the Bionic Navigator software.
4. Place the OMG Connector in a secure location on a flat surface so that patient movement will not cause the connector to fall.
5. Making sure that the Precision ETS is OFF, connect the Precision ETS to the two ports on the OMG Connector.

CAUTION	
Failure to turn off the ETS before connecting the OMG Connector may result in excess stimulation.	

Programming the Precision® ETS

1. Link the Precision Clinician Programmer (CP) to the Precision ETS per the standard procedure.
2. Use the Bionic Navigator Software to program the ETS.

Disconnecting the OMG Connector

1. Turn OFF stimulation.
2. Disconnect the ETS from the OMG Connector.
3. Disconnect the proximal end(s) of the non-Precision OR Cable Assembly(ies) from the OMG Connector.

Cleaning the OMG Connector

If necessary, the OMG Connector can be cleaned using a damp cloth or tissue. Do not use alcohol or abrasive cleaners. Do not submerge the OMG Connector in any fluids.

Customer Service - USA

In USA, to order replacement parts, or for answers to any questions you might have, please contact the Customer Service Department:

- Phone: (866) 360-4747, +1 (661) 949-4747
- Fax: (661) 362-1503

Boston Scientific Neuromodulation
25155 Rye Canyon Loop
Valencia, CA 91355, USA

Outside USA, if you have a specific question or issue, please contact your healthcare professional.

Customer Service - Outside USA

If you have any other questions or need to contact Boston Scientific, choose your locality from the contact list at the end of this document.

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All Rights Reserved