ENGLISH

OMG™ Connector – A Model SC-9305

Intended Use

The Observational Mechanical Gateway (OMG) Connector - A is used to connect the Precision® ETS to the Advanced Neuromodulation Systems OR (Model # 3009) Cable Assembly. It is intended to provide the physician with the capability to gain acute feedback from the patients to assess for any pain relief difference between the Advanced Neuromodulation Systems and the Boston Scientific System.

The OMG Connector - A is for evaluation purposes only. It is an adjunct to, but not part of, the Precision System. Full trial of the Precision System should precede permanent implant.

CAUTION 

The OMG Connector is for use only under the direct supervision of a healthcare professional. The OMG Connector may not be sent home with the patient.

Compatibility

The OMG Connector - A shall be compatible with the following Advanced Neuromodulation Systems leads that connect to the OR Cable Assembly available prior to February 2007:

Model Name	Model Number
Trial lead, 4-Channel	3046
Trial lead, 8-Channel	3086
Axxess® Trial Lead	4044, 4054
Octrode®	3183, 3186, 3189, 3181
Quattrode®	3143, 3146, 3149, 3141, 3153, 3156, 3159, 3151
Axxess®	4143, 4146, 4153, 4156
Lamitrode® 22	3222
Lamitrode® 4	3240, 3254, 3255
Lamitrode® 8	3280
Lamitrode® 44 and 44C	3244, 3262, 3263, 3245, 3264, 3265
Lamitrode® Tripole 8	3208
Lamitrode® 88 & 88c	3288, 3289

Axxess, Octrode Quattrode and Lamitrode are trademarks of Advanced Neuromodulation Systems

Directions for Use

Connecting the OMG Connector

1. Inspect the OMG Connector for damage. If damaged, do not proceed. Return the OMG Connector to Boston Scientific Neuromodulation or your healthcare professional.
2. Turn off the non-Precision External Trial Stimulator (ETS).

CAUTION 

Failure to turn off the non-Precision ETS before connecting the OMG Connector may result in excess stimulation.

3. Disconnect the proximal end(s) of the non-Precision OR Cable Assembly and insert it into the corresponding port on the OMG Connector. Plug the non-Precision OR Cable Assembly into the OMG Connector socket(s) labeled “1-L” (left) and “2-R” (right). If only a single lead is being used, connect it to “1-L”.
4. Place the OMG Connector in a secure location on a flat surface so that patient movement will not cause the connector to fall.
5. Making sure that the Precision ETS is OFF, connect the Precision ETS to the two ports on the OMG Connector.

CAUTION 

Failure to turn off the ETS before connecting the OMG Connector may result in excess stimulation.

Programming the Precision ETS

1. Link the Precision Clinician Programmer (CP) to the Precision ETS per the standard procedure.
2. Use the Bionic Navigator® Software to program the ETS.

Disconnecting the OMG Connector

1. Turn OFF stimulation.
2. Disconnect the ETS from the OMG Connector.
3. Disconnect the proximal end(s) of the non-Precision OR Cable Assembly(ies) from the OMG Connector.

Cleaning the OMG Connector

If necessary, the OMG Connector can be cleaned using a damp cloth or tissue. Do not use alcohol or abrasive cleaners. Do not submerge the OMG Connector in any fluids.

Customer Service - USA

In USA, to order replacement parts, or for answers to any questions you might have, please contact the Customer Service Department:

- Phone: (866) 360-4747, +1 (661) 949-4747
 - Fax: (661) 362-1503
- Boston Scientific Neuromodulation
25155 Rye Canyon Loop
Valencia, CA 91355, USA

Outside USA, if you have a specific question or issue, please contact your healthcare professional.

Customer Service - Outside USA

If you have any other questions or need to contact Boston Scientific, choose your locality from the contact list at the end of this document.

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