

SUMMARY

Boston Scientific implantable pacemaker and defibrillator systems utilize a **bidirectional torque wrench** to secure a lead to the pulse generator header. The torque wrench applies the appropriate force to tighten or loosen a setscrew, but also prevents application of excessive force that could cause setscrew or lead damage.

This article provides information on torque wrench operation and which wrench to use with Boston Scientific implantable devices.

Products Referenced

Model 6628 and 6942 bidirectional torque wrench
Table 1. Pacemakers, CRT-Ps, ICDs, CRT-Ds, lead adapters and extenders

Products referenced are unregistered or registered trademarks of Boston Scientific Corporation or its affiliates. All other trademarks are the property of their respective owners.

Products referenced herein may not be approved in all geographies. For comprehensive information on device operation, reference the full instructions for use found at: www.bsci.com/ifu.

CAUTION: Federal (U.S.) law restricts this device to sale by or on the order of a physician.

All graphics produced by Boston Scientific Corporation, unless otherwise noted.

CRT-D: Cardiac Resynchronization Therapy Defibrillator
CRT-P: Cardiac Resynchronization Therapy Pacemaker
ICD: Implantable Cardioverter Defibrillator
S-ICD: Subcutaneous Implantable Defibrillator

Contact Information

Americas

(Caribbean, and Central, North, and South America)

www.bostonscientific.com

Technical Services

LATITUDE® Customer Support

1.800.CARDIAC (227.3422)

+1.651.582.4000

Patient Services

1.866.484.3268

Europe, Middle East, Africa

Technical Services

+32 2 416 7222

intltechservice@bsci.com

LATITUDE Clinician Support

latitude.europe@bsci.com

Asia Pacific

Technical Services

+61 2 8063 8299

aptechservice@bsci.com

japantechservice@bsci.com

LATITUDE Customer Support

latitude.asiapacific@bsci.com

japan.latitude@bsci.com (Japan)

© 2014 by Boston Scientific Corporation or its affiliates. All Rights Reserved.

Bidirectional Torque Wrenches

Torque Wrench Operation

Boston Scientific bidirectional torque wrenches are designed for tightening and loosening setscrews on Boston Scientific pulse generators and lead accessories. Each bidirectional torque wrench is preset to apply adequate torque to the setscrew and will ratchet (click) when the setscrew is secure. Even though the wrench handle will still rotate, turning the wrench handle beyond one ratchet is not necessary because additional turns will not move the setscrew further.

NOTE: Use Boston Scientific torque wrenches with Boston Scientific devices. Using wrenches from other manufacturers may provide too much or too little torque, such that the setscrew cannot be tightened or loosened appropriately. A bidirectional torque wrench is included within pulse generator packaging or packaged individually.

WARNING: Use a new torque wrench for each procedure. Boston Scientific bidirectional torque wrenches are for single use only and should not be reused. Resterilization and reuse of the same wrench for multiple implant procedures may create a risk of contamination or alter the amount of force delivered, and could contribute to damage or difficulty operating a setscrew.

Use Appropriate Torque Wrench

Boston Scientific manufactures two torque wrench models, each preset to torque levels that match different setscrew designs. Table 1 indicates which torque wrench should be used with each Boston Scientific implantable device.

- A **Model 6628** torque wrench (white or orange) should be used on setscrews covered with white seal plugs.
- A **Model 6942** torque wrench (green) should be used on setscrews covered with clear seal plugs.

Table 1. Product Family, Seal Plug Color and Appropriate Torque Wrench Model

Product Type	Product Family	Seal Plug Color	Torque Wrench
Pacemaker	ADVANTIO™ INGENIO™ VITALIO™ FORMIO™ ALTRUA™	White Seal Plugs  	Model 6628 ¹
CRT-P	INVIVE™ INTUA™ INLIVEN™		
ICD & CRT-D	AUTOGEN™ DYNAGEN™ INOGEN™ ORIGEN™ INCEPTA™ PUNCTUA™ ENERGEN™ TELIGEN™ COGNIS™ CONFIENT™ ² LIVIAN™ ²		Model 6942
Lead Adapters & Extenders w/ white seal plugs	INSIGNIA™		
Pacemaker	CONTAK RENEWAL™ TR / TR2	Clear Seal Plugs 	Model 6942
CRT-P	CONTAK RENEWAL™ TR / TR2		
ICD & CRT-D	CONFIENT™ ² LIVIAN™ ² CONTAK RENEWAL™ VITALITY™		Model 6942
Lead Adapters & Extenders w/ clear seal plugs			

¹ A Model 6628 torque wrench may be used to loosen setscrews on devices with clear seal plugs, however, when retracting, stop turning the setscrew as soon as resistance is felt (when the setscrew is fully retracted). Any additional counterclockwise torque of this wrench could cause the setscrews to become stuck in the up position if tightened against the stop.

² Depending on geography, CONFIENT & LIVIAN devices are manufactured with all white seal plugs or all clear seal plugs.

Pacing Systems from Boston Scientific – INGENIO™, ADVANTIO™, and VITALIO™

Indications

INGENIO™, ADVANTIO™, and VITALIO™ Pacing Systems indications include: symptomatic paroxysmal or permanent second- or third-degree AV block; symptomatic bilateral bundle branch block; symptomatic paroxysmal or transient sinus node dysfunction with or without associated AV conduction disorders; bradycardia-tachycardia syndrome, to prevent symptomatic bradycardia or some forms of symptomatic tachyarrhythmias; neurovascular (vaso-vagal) syndromes or hypersensitive carotid sinus syndromes. Adaptive-rate pacing is indicated for patients exhibiting chronotropic incompetence and who may benefit from increased pacing rates concurrent with increases in minute ventilation and/or level of physical activity. Pacemakers' dual-chamber and atrial tracking modes are also indicated for patients who may benefit from maintenance of AV synchrony. Dual-chamber modes are specifically indicated for: conduction disorders that require restoration of AV synchrony, including varying degrees of AV block; VVI intolerance (eg, pacemaker syndrome) in the presence of persistent sinus rhythm or low cardiac output or congestive heart failure secondary to bradycardia.

Contraindications

Pacemakers are contraindicated for the following patients under the circumstances listed: patients who have a separate implanted cardioverter-defibrillator (ICD); use of Minute Ventilation in patients with both unipolar atrial and ventricular leads; single-chamber atrial pacing in patients with impaired AV nodal conduction; atrial tracking modes for patients with chronic refractory atrial tachyarrhythmias (atrial fibrillation or flutter), which might trigger ventricular pacing; dual-chamber and single-chamber atrial pacing in patients with chronic refractory atrial tachyarrhythmias; asynchronous pacing in the presence (or likelihood) of competition between paced and intrinsic rhythms.

Warnings

Read the product labeling thoroughly before implanting the pulse generator to avoid damage to the system. For single use only. Do not reuse, reprocess or resterilize. Always have external defibrillation protection available during implant and electrophysiologic testing. Do not use this pulse generator with another pulse generator. In response to applicable nonrecoverable or repeat fault conditions, the pulse generator will switch irreversible to Safety Core operation. Do not kink, twist, or braid leads. Do not use atrial tracking modes in patients with chronic refractory atrial tachyarrhythmias. In devices with the lead safety switch programmed to On, the lead polarity will switch to unipolar in the presence of a lead impedance of ≤ 200 or $\geq 2000 \Omega$. If programmed to a fixed atrial sensitivity value of 0.15 mV, the pulse generator may be more susceptible to electromagnetic interference. Advise patients to seek medical guidance before entering environments that could adversely affect the operation of the active implantable medical device. Do not expose a patient to MRI scanning. Do not subject a patient with an implanted pulse generator and/or lead to diathermy.

Precautions

For specific information on precautions, refer to the following sections of the product labeling: clinical considerations; sterilization and storage; implantation; device programming; environmental and medical therapy hazards; hospital and medical environments; home and occupational environments; follow up testing; explant and disposal; supplemental precautionary information. Advise patients to avoid sources of electric or magnetic interference (EMI) because EMI may cause the pulse generator to deliver inappropriate therapy or inhibit appropriate therapy.

Potential Adverse Events

Potential adverse events include, but are not limited to, the following: allergic/physical/physiologic reaction, death, erosion/migration, fibrillation or other arrhythmias, lead or accessory breakage (fracture/insulation/lead tip), hematoma/seroma, inappropriate or inability to provide therapy (pacing/sensing), infection, procedure related, and component failure. Patients may develop psychological intolerance to a pulse generator system and may experience fear of shocking, fear of device failure, or imagined shocking. In rare cases severe complications or device failures can occur.

Refer to the product labeling for specific indications, contraindications, warnings/precautions and adverse events. Rx only. (Rev. D)

Pacing Systems from Boston Scientific ALTRUA™ and INSIGNIA™

Indications

Pacemaker indications include: symptomatic paroxysmal or permanent second- or third-degree AV block; symptomatic bilateral bundle branch block; symptomatic paroxysmal or transient sinus node dysfunction with or without associated AV conduction disorders; bradycardia-tachycardia syndrome, to prevent symptomatic bradycardia or some forms of symptomatic tachyarrhythmias; neurovascular (vaso-vagal) syndromes or hypersensitive carotid sinus syndromes. Adaptive-rate pacing is indicated for patients who may benefit from increased pacing rates concurrent with increases in minute ventilation and/or level of physical activity. Pacemakers' dual-chamber and atrial tracking modes are also indicated for patients who may benefit from maintenance of AV synchrony. Dual-chamber modes are specifically indicated for: conduction disorders that require restoration of AV synchrony, including varying degrees of AV block; VVI intolerance (eg, pacemaker syndrome) in the presence of persistent sinus rhythm.

Contraindications

Pacemakers are contraindicated for the following patients under the circumstances listed: patients with unipolar pacing leads or in MV mode with an implanted ICD because it may cause unwanted delivery or inhibition of ICD therapy; use of the MV sensor in patients with only unipolar leads, because a bipolar lead is required in either the atrium or the ventricle for MV detection (INSIGNIA™ Plus, ALTRUA™ 20/40); MV mode in patients with both unipolar atrial and ventricular leads (INSIGNIA™ Ultra, ALTRUA™ 60); single-chamber atrial pacing in patients with impaired AV nodal conduction; atrial tracking modes for patients with chronic refractory atrial tachyarrhythmias, which might trigger ventricular pacing; dual-chamber and single-chamber atrial pacing in patients with chronic refractory atrial tachyarrhythmias; asynchronous pacing in the presence (or likelihood) of competition between paced and intrinsic rhythms.

Warnings

Read the product labeling thoroughly before implanting the pulse generator to avoid damage to the system. For single use only-do not resterilize devices. Inappropriate sustained high-rate pacing occurred in the PULSAR™ MAX clinical study in 5 out of 130 patients with MV ON, 4 to 14 days after implant. If sustained high-rate pacing could be of concern, consider programming a reduced Max Sensor Rate or MV to Passive. These programming recommendations are intended to assure that MV calibration is evaluated and, if necessary, recalibrated (4 → ON) when the patient and pacing system have stabilized post implant. Continued monitoring of the MV sensor performance should be performed at all follow-up visits until implant stabilization has occurred.

Precautions

For specific information on precautions, refer to the following sections of the product labeling: MV sensor calibration at implant; clinical considerations; sterilization, storage and handling; lead evaluation and connection; implantation; programming and pacemaker operation; MV initialization; environmental and medical therapy hazards; elevated pressure; explanted pacemakers. Advise patients to avoid sources of electric or magnetic interference (EMI). If the pacemaker inhibits or reverts to asynchronous operation at the programmed pacing rate or at the magnet rate while in the presence of the EMI, moving away from the source or turning it off will usually allow the pulse generator to return to its normal mode of operation.

Potential Adverse Events

Potential adverse events include, but are not limited to, the following: allergic/physical/physiologic reaction, death, erosion/migration, lead or accessory breakage (fracture/insulation/lead tip), hematoma/seroma, inappropriate or inability to provide therapy (pacing/sensing), infection, procedure related, and component failure. In rare cases severe complications or device failures can occur. Refer to the product labeling for specific indications, contraindications, warnings/precautions and adverse events. Rx only. (Rev. S)

CRT-P Systems from Boston Scientific - INVIVE™ and INTUA™

Indications

The INVIVE™ and INTUA™ Cardiac Resynchronization Therapy Pacemakers (CRT-Ps) are indicated for patients who have moderate to severe heart failure (NYHA Class III/IV) including left ventricular dysfunction (EF $\leq 35\%$) and QRS duration ≥ 120 ms and remain symptomatic despite stable, optimal pharmacologic therapy for heart failure. Atrial tracking modes are also indicated for patients who may benefit from maintenance of AV synchrony. Adaptive-rate pacing is indicated for patients exhibiting chronotropic incompetence and who would benefit from increased pacing rates concurrent with increases in physical activity.

Contraindications

These devices are contraindicated in patients who have a separate implanted cardioverter-defibrillator (ICD). Single-chamber atrial pacing is contraindicated in patients with impaired AV nodal conduction. Atrial tracking modes are contraindicated for patients with chronic refractory atrial tachyarrhythmias (atrial fibrillation or flutter), which might trigger ventricular pacing. Asynchronous pacing is contraindicated in the presence (or likelihood) of competition between paced and intrinsic rhythms.

Warnings

Read the product labeling thoroughly before implanting the pulse generator to avoid damage to the system. For single use only. Do not reuse, reprocess or resterilize. Always have external defibrillation protection available during implant and electrophysiologic testing. Do not use this pulse generator with another pulse generator. In response to applicable non-recoverable or repeat fault conditions, the pulse generator will switch irreversible to Safety Core operation. Do not kink, twist, or braid leads. Do not use atrial tracking modes in patients with chronic refractory atrial tachyarrhythmias. Do not use atrial only modes in patients with heart failure. In devices with the lead safety switch programmed to On, the lead polarity will switch to unipolar in the presence of a lead impedance of ≤ 200 or $\geq 2000 \Omega$. If programmed to a fixed atrial sensitivity value of 0.15 mV, the pulse generator may be more susceptible to electromagnetic interference. Advise patients to seek medical guidance before entering environments that could adversely affect the operation of the active implantable medical device. Do not expose a patient to MRI scanning. Do not subject a patient with an implanted pulse generator and/or lead to diathermy. Left ventricular (LV) lead dislodgment to a position near the atria can result in atrial oversensing and LV pacing inhibition.

Precautions

For specific information on precautions, refer to the following sections of the product labeling: clinical considerations; sterilization and storage; implantation; device programming; environmental and medical therapy hazards; hospital and medical environments; home and occupational environments; follow up testing; explant and disposal; and supplemental precautionary information. Advise patients to avoid sources of electric or magnetic interference (EMI) because EMI may cause the pulse generator to deliver inappropriate therapy or inhibit appropriate therapy.

Potential Adverse Events

Potential adverse events include, but are not limited to, the following: allergic/physical/physiologic reaction, death, erosion/migration, fibrillation or other arrhythmias, lead or accessory breakage (fracture/insulation/lead tip), hematoma/seroma, inappropriate or inability to provide therapy (pacing/sensing), infection, lead tip deformation and/or breakage, procedure related, and component failure. In rare cases severe complications or device failures can occur.

Refer to the product labeling for specific indications, contraindications, warnings/precautions and adverse events. Rx only. (Rev. D)

CRT-P System from Boston Scientific – CONTAK RENEWAL™ TR

Indications

The CONTAK RENEWAL™ TR Pulse Generator is indicated for patients who have moderate to severe heart failure (NYHA Class III/IV) including left ventricular dysfunction (EF ≤ 35%) and QRS duration ≥ 120 ms and remain symptomatic despite stable, optimal heart failure drug therapy (as defined in the clinical trials section in the System Guide). These devices provide atrial-ventricular tracking modes to help preserve AV synchrony and adaptive-rate pacing for patients who would benefit from adjusted pacing rates concurrent with physical activity.

Contraindications

These devices are contraindicated in patients who have a separate implanted cardioverter-defibrillator (ICD). Single-chamber atrial pacing is contraindicated in patients with impaired AV nodal conduction. Atrial tracking modes are contraindicated for patients with chronic refractory atrial tachyarrhythmias (atrial fibrillation or flutter), which might trigger ventricular pacing. Asynchronous pacing is contraindicated in the presence (or likelihood) of competition between paced and intrinsic rhythms.

Warnings

Read the product labeling thoroughly before implanting the pulse generator to avoid damage to the system. For single patient use only. Do not reuse, reprocess, or resterilize. Such damage can result in patient injury or death. Do not expose a patient to MRI device scanning. Do not expose a patient with an activated implanted pulse generator to diathermy. Do not use atrial-only modes in patients with heart failure. The clinical outcomes for patients with chronic refractory atrial tachyarrhythmias are not fully known. Safety and effectiveness studies have not been conducted. If a chronic refractory atrial tachyarrhythmia develops in a patient with these devices, do not use dual-chamber or single-chamber atrial pacing. Left ventricular (LV) lead dislodgment to a position near the atria can result in atrial oversensing and LV pacing inhibition.

Precautions

For specific information on precautions, refer to the following sections of the product labeling: clinical considerations; sterilization, storage and handling; implantation and device programming; pulse generator explant and disposal; environmental and medical therapy hazards. Advise patients to avoid sources of electromagnetic interference (EMI) because EMI may cause the pulse generator to deliver inappropriate therapy or inhibit appropriate therapy.

Potential Adverse Events

Potential adverse events include, but are not limited to, the following: allergic/physical/physiologic reaction, death, erosion/migration, fibrillation or other arrhythmias, lead or accessory breakage (fracture/insulation/lead tip), hematoma/seroma, inappropriate or inability to provide therapy (pacing/sensing), infection, lead tip deformation and/or breakage, procedure related, and component failure. In rare cases severe complications or device failures can occur.

Refer to the product labeling for specific indications, contraindications, warnings/ precautions and adverse events. Rx only. (Rev. O)

ICD Systems from Boston Scientific - DYNAGEN™ EL ICD, DYNAGEN™ MINI ICD, INOGEN™ EL ICD, INOGEN™ MINI ICD, ORIGEN™ EL ICD, and ORIGEN™ MINI ICD

Indications and Usage

Boston Scientific implantable cardioverter defibrillators (ICDs) are intended to provide ventricular antitachycardia pacing (ATP) and ventricular defibrillation for automated treatment of life-threatening ventricular arrhythmias.

Contraindications

Use of these Boston Scientific pulse generators are contraindicated for the following: patients whose ventricular tachyarrhythmias may have reversible cause, such as: digitalis intoxication, electrolyte imbalance, hypoxia, sepsis; or patients whose ventricular tachyarrhythmias have a transient cause, such as: acute myocardial infarction (MI), electrocution, drowning; or patients who have a unipolar pacemaker.

Warnings

Read this manual thoroughly before implantation to avoid damage to the pulse generator and/or lead. For single patient use only. Do not reuse, reprocess, or resterilize. Program the pulse generator Tachy Mode(s) to Off during implant, explant, or postmortem procedures to avoid inadvertent high voltage shocks. Always have external defibrillation equipment available during implant and electrophysiologic testing. Ensure that an external defibrillator and medical personnel skilled in CPR are present during post-implant device testing should the patient require external rescue. Patients should seek medical guidance before entering environments that could adversely affect the operation of the active implantable medical device, including areas protected by a warning notice that prevents entry by patients who have a pulse generator. Do not expose a patient to MRI scanning. Do not subject a patient with an implanted pulse generator and/or lead to diathermy. If desired, ensure that Patient Triggered Monitor is enabled prior to sending the patient home. Once the Patient Triggered Monitor feature has been triggered by the magnet and an EGM has been stored, or after 60 days have elapsed from the day that Store EGM was enabled, the patient should not apply the magnet. For leads that require the use of a Connector Tool, use caution handling the lead terminal when the Connector Tool is not present on the lead. Do not directly contact the lead terminal with any surgical instruments or electrical connections such as PSA (alligator) clips, ECG connections, forceps, hemostats, and clamps. Do not contact any other portion of the DF4-LLHH or DF4-LLHO lead terminal, other than the terminal pin, even when the lead cap is in place. Do not use this pulse generator with another pulse generator. This combination could cause pulse generator interaction, resulting in patient injury or a lack of therapy delivery. Do not kink, twist, or braid the lead with other leads as doing so could cause lead insulation abrasion damage or conductor damage. Do not use atrial tracking modes in patients with chronic refractory atrial tachyarrhythmias. Tracking of atrial arrhythmias could result in ventricular tachyarrhythmias.)

Precautions

For specific information on precautions, refer to the following sections of the product labeling: clinical considerations, sterilization and storage, implantation, device programming, environmental and medical therapy hazards, hospital and medical environments, home and occupational environments, follow-up testing, explant and disposal, supplemental precautionary information.

Potential Adverse Events

Potential adverse events include, but are not limited to, the following: allergic/physical/physiologic reaction, death, erosion/migration, fibrillation or other arrhythmias, lead or accessory breakage (fracture/insulation/lead tip), hematoma/seroma, inappropriate or inability to provide therapy (shocks/pacing/sensing), infection, procedure related, and component failure. Patients may develop psychological intolerance to a pulse generator system and may experience fear of shocking, fear of device failure, or imagined shocking. In rare cases severe complications or device failures can occur.

Refer to the product labeling for specific indications, contraindications, warnings/precautions and adverse events. Rx only. (Rev A)

ICD Systems from Boston Scientific - PUNCTUA™, ENERGEN™, and INCEPTA™

ICD Indications and Usage

PUNCTUA™, ENERGEN™, and INCEPTA™ ICDs are intended to provide ventricular antitachycardia pacing and ventricular defibrillation for automated treatment of life-threatening ventricular arrhythmias.

Contraindications

Use of these ICD systems are contraindicated in: Patients whose ventricular tachyarrhythmias may have reversible cause, such as 1) digitalis intoxication, 2) electrolyte imbalance, 3) hypoxia, or 4) sepsis, or whose ventricular tachyarrhythmias have a transient cause, such as 1) acute myocardial infarction, 2) electrocution, or 3) drowning. Patients who have a unipolar pacemaker.

Warnings

Read the product labeling thoroughly before implanting the pulse generator to avoid damage to the ICD system. For single patient use only. Do not reuse, reprocess, or resterilize. Program the pulse generator ventricular Tachy Mode to Off during implant, explant or post-mortem procedures. Always have external defibrillator protection available during implant and electrophysiologic testing. Ensure that an external defibrillator and medical personnel skilled in cardiopulmonary resuscitation (CPR) are present during post-implant device testing. Patients should seek medical guidance before entering environments that could adversely affect the operation of the active implantable medical device, including areas protected by a warning notice that prevents entry by patients who have a pulse generator. Do not expose a patient to MRI scanning. Do not subject a patient with an implanted pulse generator to diathermy. Do not use atrial tracking modes in patients with chronic refractory atrial tachyarrhythmias. Do not use this pulse generator with another pulse generator. Do not kink, twist or braid lead with other leads. For Patient Triggered Monitor (PTM) feature, make sure the feature is enabled prior to sending the patient home with a magnet. Once the PTM feature has been triggered and the magnet response programming is set to inhibit therapy, the patient should not reapply the magnet. For DF4-LLHH or DF4-LLHO leads, use caution handling the lead terminal when the Connector Tool is not present on the lead and do not directly contact the lead terminal with any surgical instruments or electrical connections such as PSA (alligator) clips, ECG connections, forceps, hemostats, and clamps. Do not contact any other portion of the DF4-LLHH or DF4-LLHO lead terminal, other than the terminal pin even when the lead cap is in place.

Precautions

For specific information on precautions, refer to the following sections of the product labeling: clinical considerations; sterilization and storage; implantation; device programming; environmental and medical therapy hazards; hospital and medical environments; home and occupational environments follow-up testing; explant and disposal; supplemental precautionary information. Advise patients to avoid sources of electromagnetic interference (EMI).

Potential Adverse Events

Potential adverse events from implantation of the ICD system include, but are not limited to, the following: allergic/physical/physiologic reaction, death, erosion/migration, fibrillation or other arrhythmias, lead or accessory breakage (fracture/insulation/lead tip), hematoma/seroma, inappropriate or inability to provide therapy (shocks/pacing/sensing), infection, procedure related, psychologic intolerance to an ICD system – patients susceptible to frequent shocks despite antiarrhythmic medical management/imagined shocking, and component failure. In rare cases severe complications or device failures can occur.

Refer to the product labeling for specific indications, contraindications, warnings/ precautions and adverse events. Rx only. (Rev C)

ICD Systems from Boston Scientific - TELIGEN™

ICD Indications and Usage

ICDs are intended to provide ventricular antitachycardia pacing and ventricular defibrillation for automated treatment of life-threatening ventricular arrhythmias. ICDs (i.e. Vitality AVT) with atrial therapies are also intended to provide atrial antitachycardia pacing and atrial defibrillation treatment in patients who have or are at risk of developing atrial tachyarrhythmias.

Contraindications

Use of ICD systems are contraindicated in: Patients whose ventricular tachyarrhythmias may have reversible cause, such as 1) digitalis intoxication, 2) electrolyte imbalance, 3) hypoxia, or 4) sepsis, or whose ventricular tachyarrhythmias have a transient cause, such as 1) acute myocardial infarction, 2) electrocution, or 3) drowning. Patients who have a unipolar pacemaker.

Warnings

Read the product labeling thoroughly before implanting the pulse generator to avoid damage to the ICD system. For single patient use only. Do not reuse, reprocess, or resterilize. Program the pulse generator ventricular Tachy Mode to Off during implant, explant or post-mortem procedures. Always have external defibrillator protection available during implant and electrophysiologic testing. Ensure that an external defibrillator and medical personnel skilled in cardiopulmonary resuscitation (CPR) are present during post-implant device testing. Patients should seek medical guidance before entering environments that could adversely affect the operation of the active implantable medical device, including areas protected by a warning notice that prevents entry by patients who have a pulse generator. Do not expose a patient to MRI scanning. Do not subject a patient with an implanted pulse generator to diathermy. Do not use atrial tracking modes (or an AVT device) in patients with chronic refractory atrial tachyarrhythmias. Do not use this pulse generator with another pulse generator. Do not kink, twist or braid lead with other leads. For Patient Triggered Monitor (PTM) feature, make sure the feature is enabled prior to sending the patient home with a magnet. Once the PTM feature has been triggered and the magnet response programming is set to inhibit therapy, the patient should not reapply the magnet.

Precautions

For specific information on precautions, refer to the following sections of the product labeling: clinical considerations; sterilization and storage; implantation; device programming; environmental and medical therapy hazards; hospital and medical environments; home and occupational environments follow-up testing; explant and disposal; supplemental precautionary information. Advise patients to avoid sources of electromagnetic interference (EMI).

Potential Adverse Events

Potential adverse events from implantation of the ICD system include, but are not limited to, the following: allergic/physical/physiologic reaction, death, erosion/migration, fibrillation or other arrhythmias, lead or accessory breakage (fracture/insulation/lead tip), hematoma/seroma, inappropriate or inability to provide therapy (shocks/pacing/sensing), infection, procedure related, psychologic intolerance to an ICD system – patients susceptible to frequent shocks despite antiarrhythmic medical management/imagined shocking, and component failure. In rare cases severe complications or device failures can occur.

Refer to the product labeling for specific indications, contraindications, warnings/ precautions and adverse events. Rx only. (Rev. S)

CRT-D Systems from Boston Scientific – DYNAGEN™, INOGEN™, and ORIGEN™

Indications and Usage

These Boston Scientific Cardiac Resynchronization Therapy Defibrillators (CRT-Ds) are indicated for patients with heart failure who receive stable optimal pharmacologic therapy (OPT) for heart failure and who meet any one of the following classifications: Moderate to severe heart failure (NYHA Class III-IV) with EF \leq 35% and QRS duration \geq 120 ms; or left bundle branch block (LBBB) with QRS duration \geq 130 ms, EF \leq 30%, and mild (NYHA Class II) ischemic or nonischemic heart failure or asymptomatic (NYHA Class I) ischemic heart failure.

Contraindications

There are no contraindications for this device.

Warnings

Read this manual thoroughly before implantation to avoid damage to the pulse generator and/or lead. For single patient use only. Do not reuse, reprocess, or resterilize. Always have external defibrillation equipment available during implant and electrophysiologic testing. Ensure that an external defibrillator and medical personnel skilled in CPR are present during postimplant device testing should the patient require external rescue. Advise patients to seek medical guidance before entering environments that could adversely affect the operation of the active implantable medical device, including areas protected by a warning notice that prevents entry by patients who have a pulse generator. Do not expose a patient to MRI scanning. Do not subject a patient with an implanted pulse generator and/or lead to diathermy. Do not use atrial tracking modes in patients with chronic refractory atrial tachyarrhythmias. Do not use atrial-only modes in patients with heart failure. Left ventricular lead dislodgement to a position near the atria can result in atrial oversensing and left ventricular pacing inhibition. Physicians should use medical discretion when implanting this device in patients who present with slow VT. Do not kink, twist, or braid the lead with other leads. Do not use defibrillation patch leads with the pulse generator system. Do not use this pulse generator with another pulse generator. Ensure that Patient Triggered Monitor (PTM) is enabled prior to sending the patient home by confirming the magnet response is programmed to Store EGM. Once the PTM feature has been triggered and the magnet response set to Inhibit therapy the patient should not reapply the magnet. For DF4-LLHH or DF4-LLHO leads, use caution handling the lead terminal when the Connector Tool is not present on the lead and do not directly contact the lead terminal with any surgical instruments or electrical connections such as PSA (alligator) clips, ECG connections, forceps, hemostats, and clamps. Do not contact any other portion of the DF4-LLHH or DF4-LLHO lead terminal, other than the terminal pin, even when the lead cap is in place. Do not contact any other portion of the IS4-LLLL lead terminal, other than the terminal pin, even when the lead cap is in place.

Precautions

For specific information on precautions, refer to the following sections of the product labeling: clinical considerations, sterilization and storage, implantation, device programming, hospital and medical environments, home and occupational environments, environmental and medical therapy hazards, follow-up testing, explant and disposal, supplemental precautionary information. Advise patients to avoid sources of EMI because EMI may cause the pulse generator to deliver inappropriate therapy or inhibit appropriate therapy.

Potential Adverse Events

Potential adverse events include, but are not limited to, the following: allergic/physical/physiologic reaction, death, erosion/migration, fibrillation or other arrhythmias, lead or accessory breakage (fracture/insulation/lead tip), hematoma/seroma, inappropriate or inability to provide therapy (shocks/pacing/sensing), infection, procedure related, and component failure. Patients may develop psychological intolerance to a pulse generator system and may experience fear of shocking, fear of device failure, or imagined shocking. In rare cases severe complications or device failures can occur.

Refer to the product labeling for specific indications, contraindications, warnings/precautions and adverse events. Rx only. (Rev A)

CRT-D Systems from Boston Scientific – PUNCTUA™, ENERGEN™, INCEPTA™

Indications and Usage

The PUNCTUA™, ENERGEN™, and INCEPTA™ Cardiac Resynchronization Therapy Defibrillators (CRT-Ds) are indicated for patients with heart failure who receive stable optimal pharmacologic therapy (OPT) for heart failure and who meet any one of the following classifications:

- Moderate to severe heart failure (NYHA Class III-IV) with EF \leq 35% and QRS duration \geq 120 ms
- Left bundle branch block (LBBB) with QRS \geq 130 ms, EF \leq 30%, and mild (NYHA Class II) ischemic or nonischemic heart failure or asymptomatic (NYHA Class I) ischemic heart failure

Contraindications

There are no contraindications for this device.

Warnings

Read the product labeling thoroughly before implanting the pulse generator to avoid damage to the system. For single patient use only. Do not reuse, reprocess, or resterilize. Program the pulse generator Tachy Mode to Off during implant, explant or postmortem procedures. Always have external defibrillator protection available during implant and electrophysiologic testing. Ensure that an external defibrillator and medical personnel skilled in CPR are present during post-implant device testing. Advise patients to seek medical guidance before entering environments that could adversely affect the operation of the active implantable medical device, including areas protected by a warning notice that prevents entry by patients who have a pulse generator. Do not expose a patient to MRI scanning. Do not subject a patient with an implanted pulse generator to diathermy. Do not use atrial-tracking modes in patients with chronic refractory atrial tachyarrhythmias. Do not use atrial-only modes in patients with heart failure. LV lead dislodgment to a position near the atria can result in atrial oversensing and LV pacing inhibition. Physicians should use medical discretion when implanting this device in patients who present with slow VT. Do not kink, twist or braid the lead with other leads. Do not use defibrillation patch leads with the CRT-D system. Do not use this pulse generator with another pulse generator. For Patient Triggered Monitor (PTM) feature, make sure the feature is enabled prior to sending the patient home with a magnet. Once the PTM feature has been triggered and the magnet response programming is set to inhibit therapy, the patient should not reapply the magnet. For DF4-LLHH or DF4-LLHO leads, use caution handling the lead terminal when the Connector Tool is not present on the lead and do not directly contact the lead terminal with any surgical instruments or electrical connections such as PSA (alligator) clips, ECG connections, forceps, hemostats, and clamps. Do not contact any other portion of the DF4-LLHH or DF4-LLHO lead terminal, other than the terminal pin even when the lead cap is in place.

Precautions

For specific information on precautions, refer to the following sections of the product labeling: clinical considerations; sterilization and storage; implantation; device programming; follow-up testing; explant and disposal; environmental and medical therapy hazards; hospital and medical environments; home and occupational environments; and supplemental precautionary information. Advise patients to avoid sources of electromagnetic interference (EMI) because EMI may cause the pulse generator to deliver inappropriate therapy or inhibit appropriate therapy.

Potential Adverse Events

Potential adverse events from implantation of the CRT-D system include, but are not limited to, the following: allergic/physical/physiologic reaction, death, erosion/migration, fibrillation or other arrhythmias, lead or accessory breakage (fracture/insulation/lead tip), hematoma/seroma, inappropriate or inability to provide therapy (shocks/pacing/sensing), infection, procedure related, and component failure. Patients may develop psychological intolerance to a pulse generator system and may experience fear of shocking, fear of device failure, or imagined shocking. In rare cases severe complications or device failures can occur.

Refer to the product labeling for specific indications, contraindications, warnings/precautions and adverse events. Rx only. (Rev. C)

CRT-D Systems from Boston Scientific – COGNIS™

Indications and Usage

These Boston Scientific Cardiac Resynchronization Therapy Defibrillators (CRT-Ds) are indicated for patients with heart failure who receive stable optimal pharmacologic therapy (OPT) for heart failure and who meet any one of the following classifications:

- Moderate to severe heart failure (NYHA Class III-IV) with EF \leq 35% and QRS duration \geq 120 ms
- Left bundle branch block (LBBB) with QRS \geq 130 ms, EF \leq 30%, and mild (NYHA Class II) ischemic or nonischemic heart failure or asymptomatic (NYHA Class I) ischemic heart failure

Contraindications

There are no contraindications for this device.

Warnings

Read the product labeling thoroughly before implanting the pulse generator to avoid damage to the system. For single patient use only. Do not reuse, reprocess, or resterilize. Program the pulse generator Tachy Mode to Off during implant, explant or postmortem procedures. Always have sterile external and internal defibrillator protection available during implant and electrophysiologic testing. Ensure that an external defibrillator and medical personnel skilled in CPR are present during post-implant device testing. Advise patients to seek medical guidance before entering environments that could adversely affect the operation of the active implantable medical device, including areas protected by a warning notice that prevents entry by patients who have a pulse generator. Do not expose a patient to MRI scanning. Do not subject a patient with an implanted pulse generator to diathermy. Do not use atrialtracking modes in patients with chronic refractory atrial tachyarrhythmias. Do not use atrial-only modes in patients with heart failure. LV lead dislodgment to a position near the atria can result in atrial oversensing and LV pacing inhibition. Physicians should use medical discretion when implanting this device in patients who present with slow VT. Do not kink, twist or braid the lead with other leads. Do not use defibrillation patch leads with the CRT-D system. Do not use this pulse generator with another pulse generator. For Patient Triggered Monitor (PTM) feature, make sure the feature is enabled prior to sending the patient home with a magnet. Once the PTM feature has been triggered and the magnet response programming is set to inhibit therapy, the patient should not reapply the magnet.

Precautions

For specific information on precautions, refer to the following sections of the product labeling: clinical considerations; sterilization, storage and handling; implant and device programming; follow-up testing; explant and disposal; environmental and medical therapy hazards; hospital and medical environments; home and occupational environments. Advise patients to avoid sources of electromagnetic interference (EMI) because EMI may cause the pulse generator to deliver inappropriate therapy or inhibit appropriate therapy.

Potential Adverse Events

Potential adverse events from implantation of the CRT-D system include, but are not limited to, the following: allergic/physical/physiologic reaction, death, erosion/migration, fibrillation or other arrhythmias, lead or accessory breakage (fracture/insulation/lead tip), hematoma/seroma, inappropriate or inability to provide therapy (shocks/pacing/sensing), infection, procedure related, and component failure. Patients may develop psychological intolerance to a pulse generator system and may experience fear of shocking, fear of device failure, or imagined shocking. In rare cases severe complications or device failures can occur.

Refer to the product labeling for specific indications, contraindications, warnings/precautions and adverse events. Rx only. (Rev. S)